strip (Urine)

# MPD-U12

The MPD Rapid Test Strip (Urine) is a rapid visual immunoassay for the qualitative, presumptive detection of MPD in human urine specimens at the cut-off concentrations listed below:

Parameter	Calibrator	Cut-off (ng/mL)
MPD	Methylphenidate	150

## INTRODUCTION

Methylphenidate is a psychostimulant drug approved for treatment of ADHD or attention-deficit hyperactivity disorder, postural orthostatic tachycardia syndrome and narcolepsy. Methylphenidate acts primarily to inhibit the reuptake of dopamine and to a lesser extent norepinephrine thereby retaining these hormones longer which increases the levels of these neurotransmitters in the brain.

Methylphenidate possesses some structural and pharmacological similarities to cocaine, though methylphenidate is less potent and longer in duration. Methylphenidate has some potential for abuse due to its action on dopamine transporters, although individuals with ADHD are unable to get any recreational use. Methylphenidate, like other stimulants, increases dopamine levels in the brain, but at therapeutic doses this increase is slow, and thus euphoria only rarely occurs even when it is administered intravenously. The abuse and addiction potential of methylphenidate is therefore significantly lower than that of cocaine. The abuse potential is increased when methylphenidate is crushed and insufflated (snorted), or when it is injected, producing effects somewhat similar to cocaine. Cocaine-like effects can also occur with very large doses taken orally. However, the dose that produces euphoric effects varies among individuals

#### PRINCIPLE

The MPD Rapid Test Strip (Urine) detects Methylphenidate through visual interpretation of color development on the strip. Drug conjugates are immobilized on the test region of the membrane. During testing, the specimen reacts with antibodies conjugated to colored particles and precoated on the sample pad. The mixture then migrates through the membrane by capillary action, and interacts with reagents on the membrane. If there are insufficient drug molecules in the specimen, the antibody-colored particle conjugate will bind to the drug conjugates, forming a colored band at the test region of the membrane. Therefore, a colored band appears in the test region when the urine is negative for the drug. If drug molecules are present in the urine above the cut-off concentration of the test, they compete with the immobilized drug conjugate on the test region for limited antibody binding sites. This will prevent attachment of the antibody-colored particle conjugate to the test region. Therefore, the absence of a colored band at the test region indicates a positive result. The appearance of a colored band at the control region serves as a procedural control, indicating that the proper volume of specimen has been added and membrane wicking has occurred.

#### REAGENTS

Each test consists of a reagent strip mounted in a plastic housing. The amount of each antigen and/or antibody coated on the strip is less than 0.001 mg for antigen conjugates and goat anti-rabbit IgG antibodies, and less than 0.0015 mg for antibody components.

The control zone of each test contains goat anti-rabbit IgG antibody. The test zone of each test contains drug-bovine protein antigen conjugate, and the conjugate pad of each test contains monoclonal anti-drug antibody and rabbit antibody-colored particle complex.

## MATERIALS

## Materials Provided

Test strips

· Package insert

## Materials Required but Not provided

Positive and negative controls

Timer

Centrifuge

## PRECAUTIONS

- For Forensic Use only
- · Read the entire procedure carefully prior to testing.
- . Do not use after the expiration date indicated on the package. Do not use the test if the canister is
- · This kit contains products of animal origin. Certified knowledge of the origin and/or sanitary state of the animals does not completely guarantee the absence of transmissible pathogenic agents. It is therefore, recommended that these products be treated as potentially infectious, and handled by observing usual safety precautions (e.g., do not ingest or inhale).
- Avoid cross-contamination of specimens by using a new specimen collection container for each specimen obtained
- Do not eat, drink or smoke in the area where specimens and kits are handled. Handle all specimens as if they contain infectious agents. Observe established precautions against microbiological hazards throughout the procedure and follow standard procedures for the proper disposal of specimens. Wear protective clothing such as laboratory coats, disposable gloves and eye protection when specimens are assayed.
- Humidity and temperature can adversely affect results.
- Used testing materials should be discarded in accordance with local regulations.

#### STORAGE AND STABILITY

The kit should be stored at 2-30°C until the expiry date printed on the canister.

- · The test must remain in the closed canister until use.
- Do not freeze.
- Kits should be kept out of direct sunlight.
- · Care should be taken to protect the components of the kit from contamination. Do not use if there is evidence of microbial contamination or precipitation. Biological contamination of dispensing equipment, containers or reagents can lead to false results.


## SPECIMEN COLLECTION AND STORAGE


- . The MPD Rapid Test Strip (Urine) is intended for use with human urine specimens only.
- · Urine collected at any time of the day may be used.
- Urine specimens must be collected in clean, dry containers.
- Turbid specimens should be centrifuged, filtered, or allowed to settle and only the clear supernatant should be used for testing.
- · Perform testing immediately after specimen collection. Do not leave specimens at room temperature for prolonged periods. Urine specimens may be stored at 2-8°C for up to 2 days. For long term storage, specimens should be kept below -20°C.
- Bring specimens to room temperature prior to testing. Frozen specimens must be completely thawed and mixed well prior to testing. Avoid repeated freezing and thawing of specimens.
- · If specimens are to be shipped, pack them in compliance with all applicable regulations for transportation of etiological agents.


## PROCEDURE

#### Bring tests, specimens, and/or controls to room temperature (15-30°C) before use.

- 1. Remove one strip from the canister, and use it as soon as possible. For best results, the assay should be performed within one hour. Canisters should be closed tightly after removing strips.
- Hold the strip by the end, where the product name is printed. To avoid contamination, do not touch the
- Hold the strip vertically, dip the test strip in the urine specimen for at least 10-15 seconds. Do not immerse past the maximum line (MAX) on the test strip.
- 4. After the test has finished running, remove the strip from the specimen and place it on a non-absorbent flat surface. Start the timer and wait for the colored band(s) to appear. The result should be read at 5 minutes. Do not interpret the result after 10 minutes.

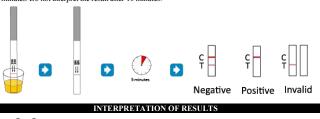

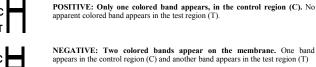

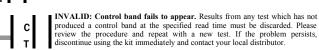

# NOTE:

- 1. The intensity of color in the test region (T) may vary depending on the concentration of analytes present in the specimen. Therefore, any shade of color in the test region should be considered negative. Note that this is a qualitative test only, and cannot determine the concentration of analytes in the specimen.
- 2. Insufficient specimen volume, incorrect operating procedure or expired tests are the most likely reasons for control band failure.

#### QUALITY CONTROL

- · Internal procedural controls are included in the test. A colored band appearing in the control region (C) is considered an internal positive procedural control, confirming sufficient specimen volume and correct procedural technique.
- · External controls are not supplied with this kit. It is recommended that positive and negative controls be tested as a good laboratory practice to confirm the test procedure and to verify proper

## LIMITATIONS OF THE TEST

- The MPD Rapid Test Strip (Urine) is for forensic use only, and should be only used for the qualitative
- This assay provides a preliminary analytical test result only. A more specific alternative chemical method must be used in order to obtain a confirmed analytical result. Gas chromatography/mass

- spectrometry (GC/MS) has been established as the preferred confirmatory method by the National
- Institute on Drug Abuse (NIDA). Clinical consideration and professional judgment should be applied to any test result, particularly when preliminary positive results are indicated.
- There is a possibility that technical or procedural errors as well as other substances and factors may interfere with the test and cause false results.
- Adulterants, such as bleach and/or alum, in urine specimens may produce erroneous results regardless of the analytical method used. Therefore, please preclude the possibility of urine adulteration prior to
- A positive result indicates the presence of a Methylphenidate, and does not indicate or measure intoxication. A negative result does not at any time rule out the presence of Methylphenidate in urine, as they may be present below the minimum detection level of the test.
- This test does not distinguish between Methylphenidate and certain medications.

## PERFORMANCE CHARACTERISTICS A. Accuracy

The accuracy of The MPD Rapid Test Strip (Urine) was compared and checked against commercially available tests with a threshold value at the same cut-off levels. Urine samples taken from volunteers claiming to be non-users were examined under both tests. The results were >97% in agreement.

#### B. Reproducibility

The reproducibility of The MPD Rapid Test Strip (Urine) was verified by blind tests performed at four different locations. Samples with Methylphenidate concentrations at 50% of the cut-off were all determined to be negative, while samples with Methylphenidate concentrations at 200% of the cut-off were all determined to be positive.

#### C. Precision

Test precision was determined by blind tests with control solutions. Controls with MPD concentrations at 50% of the cut-off yielded negative results, and controls with MPD concentrations at 150% of the cut-off yielded positive results.

#### D. Specificity

The following tables list the concentrations of compounds (ng/mL) above which The MPD Rapid Test Strip (Urine) identified positive results at 5 minutes.

MPD related compounds	Concentration (ng/mL)
Methylphenidate	150
Ritaline acid	5000

The following compounds yielded negative results up to a concentration of 100 ug/mI :

The following compounds yielded negative results up to a concentration of 100 μg/mL:				
(-)-Ephedrine	Chlorpheniramine	Oxalic Acid		
(+)-Naproxen	Creatine	Penicillin-G		
(+/-)-Ephedrine	Dextromethorphan	Pheniramine		
4-Dimethyllaminoantiyrine	Dextrorphan tartrate	Phenothiazine		
Acetaminophen	Dopamine	Procaine		
Acetone	Erythromycin	Protonix		
Albumin	Ethanol	Pseudoephedrii		
Amitriptyline	Furosemide	Quinidine		
Ampicillin	Glucose	Ranitidine		
Aspartame	Guaiacol Glyceryl Ether	Sertraline		
Aspirin	Hemoglobin	Tyramine		
Benzocaine	Imipramine	Trimeprazine		
Bilirubin	(+/-)-Isoproterenol	Venlafaxine		
b-Phenylethyl-amine	Methadone	Ibuprofen		
Caffeine	Vitamin C (Ascorbic Acid)	Lidocaine		
Chloroquine				

## LITERATURE REFERENCES

- 1. Baselt RC. Disposition of Toxic Drugs and Chemicals in Man. 2nd ed. Davis: Biomedical Publications: 1982
- 2. Hawks RL, Chiang CN, eds. Urine Testing for Drugs of Abuse. Rockville: Department of Health and Human Services, National Institute on Drug Abuse; 1986.
- Substance Abuse and Mental Health Services Administration. Mandatory Guidelines for Federal Workplace Drug Testing Programs. 53 Federal Register; 1988.
- 4. McBay AJ. Drug-analysis technology--pitfalls and problems of drug testing. Clin Chem. 1987 Oct; 33 (11 Suppl): 33B-40B.
- 5. Gilman AG, Goodman LS, Gilman A, eds. Goodman and Gilman's The Pharmacological Basis of Therapeutics. 6th ed. New York: Macmillan; 1980

GLOSSARY OF SYMBOLS					
REF	Catalog number	4	Temperature limitation		
I	Consult instructions for use	LOT	Batch code		
IVD	In vitro diagnostic medical device	8	Use by		
-	Manufacturer	2	Do not reuse		